

PetLink+

Wirelessly Links to Our Smartphone App

QUICK REFERENCE GUIDE



Dear PetTest Glucose Monitoring System Owner:

Thank you for choosing the PetTest Blood Glucose Monitoring System. This system is specifically calibrated for dogs and cats; therefore, it is NOT intended for human use. This Quick Reference Guide provides an overview of the steps for running a blood glucose test on your pet. For setup information and complete instructions, refer to the User Manual*. Please read this Guide and the User Manual carefully before using the system. Use only under the direction of a veterinary professional between vet visits.

Distributed By
Diabetic Supply of Suncoast, Inc.
PO Box 2102, Vega Alta, PR 00692
1-866-373-2824
(24 hours a day / 7 days a week)
www.ShopPetTest.com

Follow Us:
facebook.com/PetDiabetes

User Manual:
shoppettest.com/manuals

*The User Manual is sometimes referred to as the Owner's Manual.




Items for Testing



The PetTest Blood Glucose Monitoring System is specifically calibrated for dogs and cats to give accurate readings. Just 3 easy steps to monitor your pet's blood glucose levels! **Successful blood glucose testing requires patience and repetition to develop a successful technique and consistent results.**

- 1** Insert test strip with contact points into meter as pictured in this photo.
- 2** Carefully but firmly press strip into meter until snug.
- 3** Completely fill the test strip window with blood.

How to properly fill a test strip:

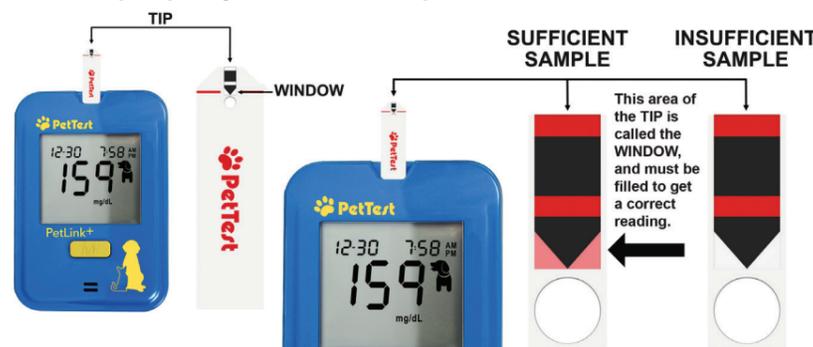

How to Initially Connect BLE-enabled PetLink+ Meter to PetTest Digital Companion App:

1. Turn on the meter by tapping the "M" button.
2. Push and hold the "M" button for 3 seconds to turn on wireless transmission. The "BLE" text on the meter will indicate that wireless transmission is turned on/enabled.
3. Open the PetTest Digital Companion App on your smartphone, then go into "Settings", and press "Connection". You will find a 5 digit pairing number to the right of "PetTest" which should be the same as what is shown on the PetLink+ LCD screen above "BLE". Connect the meter to your app wirelessly by tapping the broken grey link/chain icon located on the top right on your app screen. Your meter is paired when you see the link/chain icon interlocked and turns from grey to maroon. Hit the "Back" button (or swipe left) to return to the main "Settings" menu.



4. **Settings:** You can use the app's "Settings" function listed under the "BLE Data Sync" heading to set your meter's date/time, alarm(s), pet species, etc. You can also elect to "Clear Memory", which will delete all meter memory/entries (but not the logs in the app). See User Manual if you need to change units (from mg/dL to mmol/L or vice versa).
5. **Importing meter data to the app:** Press the "Import Data" button, and data will start transmitting to the app. During transmission, it will show a "Loading" animation on the app. When data transmission is finished, the app will show "Data Import Completed". Tap "Records" button on the bottom to see your newly-imported readings graphically displayed, or "Log" button to see individual BG entries.
6. **After testing, the meter will turn on the BLE function automatically.** The meter screen will display the 5 digit pairing number, and then you can press the "Connection" button to connect in the app. If you don't want to connect to app at that time, simply pull the strip out of the meter. This will turn off the meter, or, if nothing is done, the meter will turn off 5 minutes later automatically.